


Quick Reference Guide

Masimo SafetyNet™ App

WARNING: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

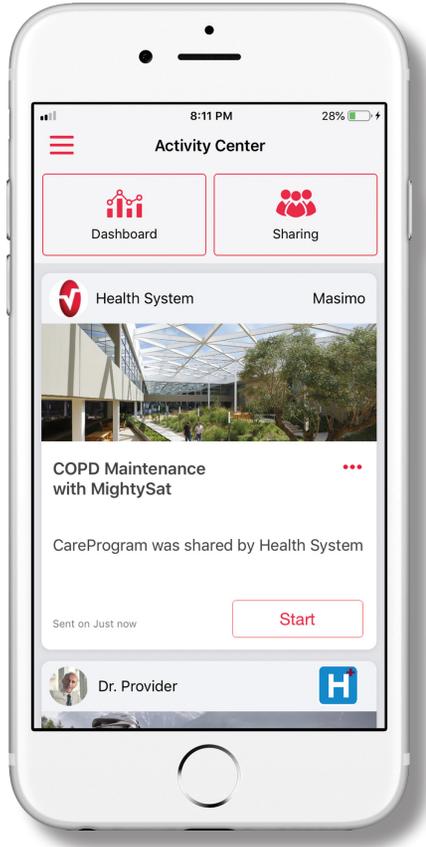

> Download and install the Masimo SafetyNet App from:

The App Store



Google Play



> Launch the app and follow the step-by-step instructions to create an account, engage in a shared CareProgram™, and begin collecting and sharing data.

For further assistance or questions visit:
www.masimo.com/masimo-safetynet



Not intended to be used in the prevention, diagnosis, or treatment of any disease. The use of the trademark PATIENT SAFETYNET is under license from University HealthSystem Consortium. All other trademarks and registered trademarks are property of their respective owners.

Log In, Sign Up, Notifications

> Ensure notifications are allowed for proper app function.

> Sign Up

- If you received a CareProgram from a health provider, you must sign in with the email or phone number used to send your invitation.
- Sign up for an account using email, phone, or with Google single-sign on as your username.



This app allows you to securely share your health data with anyone.

You can also receive care instructions from your doctor through this app.


Log In Using Google

Or Log In Using Doctella ID

Sign Up


Log In

iOS

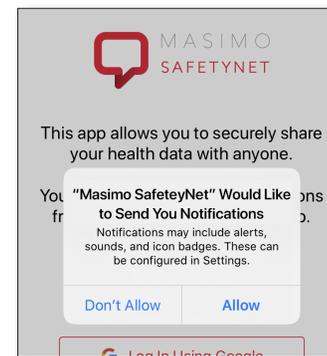

ALLOW NOTIFICATIONS


Android

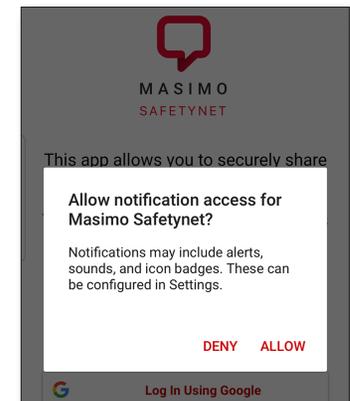

DENY ALLOW


Health Permissions

iOS

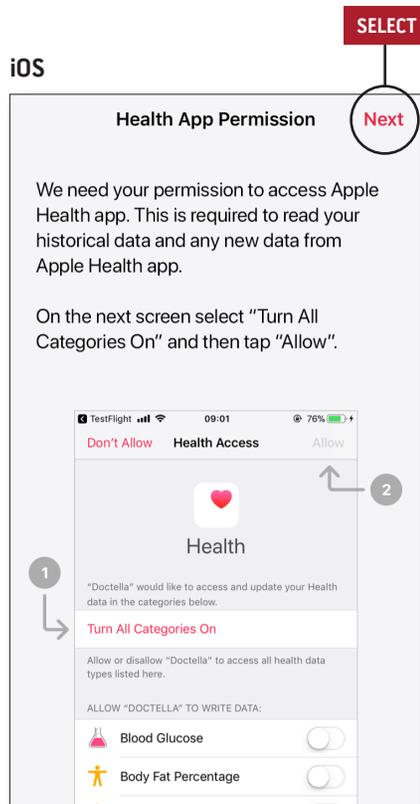


- > **Follow the on-screen instructions:**
To allow Masimo SafetyNet to access and gather health data from the device.

Android

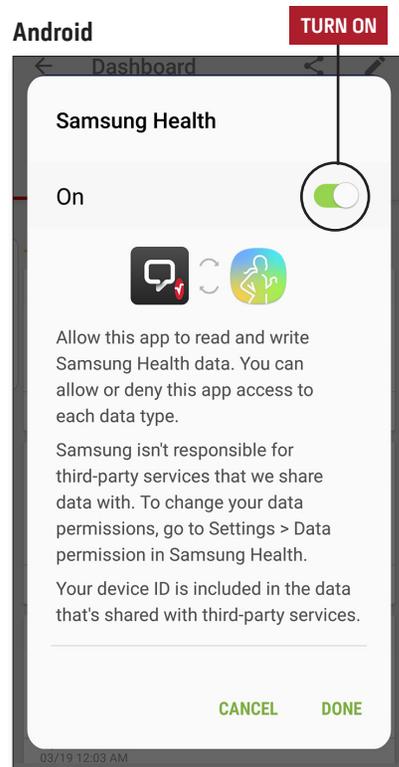


Activity Center

MENU

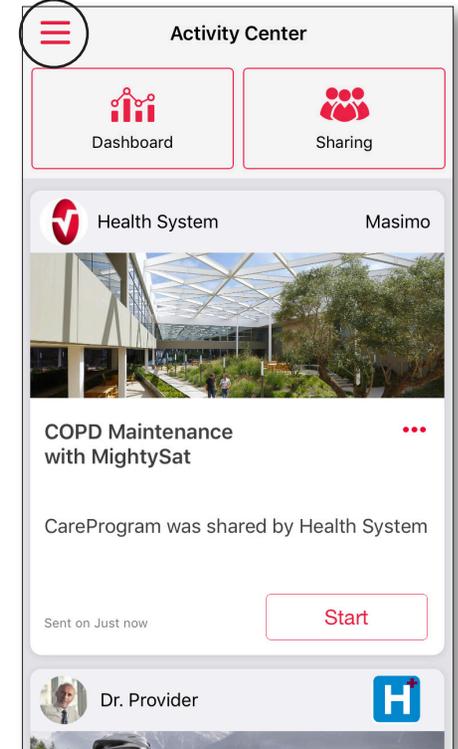

- > **Menu**
 - View the App Menu.
- > **Dashboard**
 - View the Dashboard screen, which shows the user's health data.
 - View health data shared by other users.
- > **Sharing**
 - Displays the healthcare providers, caregivers, and family members with whom the user is currently sharing data.
 - Share data with another healthcare provider, caregiver, or family member.
- > **CareProgram**
 - Allows user to view or delete a CareProgram.


Menu

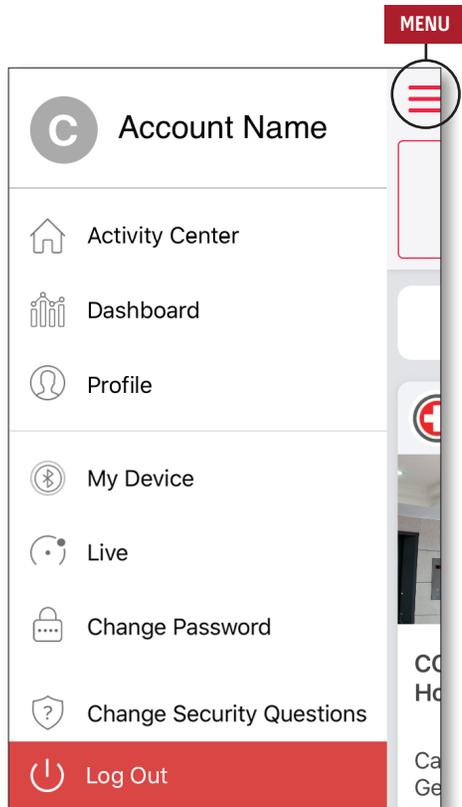

- > **Activity Center**
 - Displays the Activity Center screen.
- > **Dashboard**
 - Displays the Dashboard screen.
- > **Profile**
 - Displays the user profile information.
 - Allows the user to modify the profile information.
- > **My Device**
 - Allows the user to add a device for monitoring.
- > **Live**
 - Allows the user to view data from the connected monitoring device.
- > **Change Password/Change Security Questions**
 - Allows the user to change the password and/or security questions in case of lost password.
- > **Notification Settings**
 - Allows the user to change the app notification and reminder settings.
- > **Settings**
 - Displays the user ID.
 - Allows user to enable or disable Touch ID or Face ID to log into app.
 - Displays app information.


- > **Feedback**
 - Allows the user to send an email to Masimo SafetyNet support.
- > **Rate on App Store**
 - Allows the user to rate the app on the app store.

Dashboard

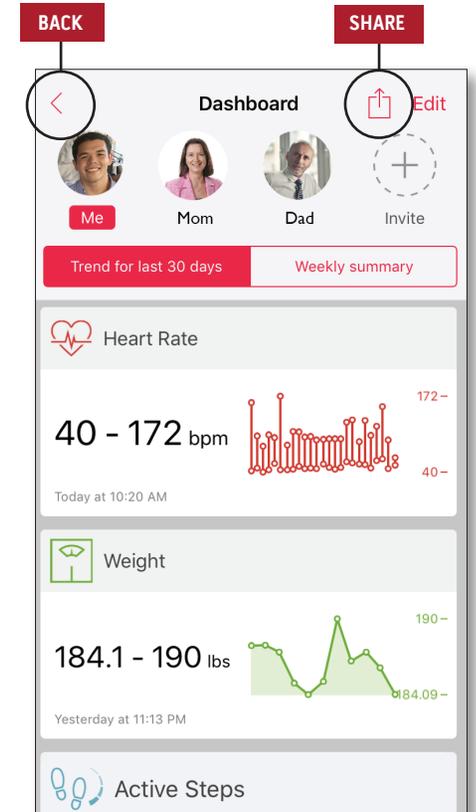


- > **Invite**
 - Allows the user to invite others to share information through the app.
- > **Sharing**
 - Displays the healthcare providers, caregivers, and family members with whom the user is currently sharing data.
- > **Edit**
 - Allows the user to rearrange data on the Dashboard.
- > **View data for the last 30 days or a weekly summary.**
 - Select a parameter card to view trends, date ranges, and additional data.

CareProgram

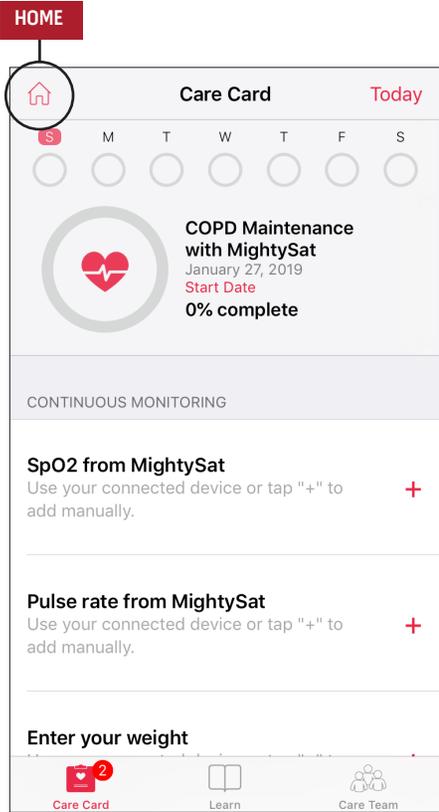

- > **Care Card**
 - View the Care Card data.
- > **Home**
 - Return to the Activity Center.
- > **Learn**
 - View additional information provided by a caregiver.
- > **Care Team**
 - View the list of shared healthcare providers, caregivers, and family members.

Sharing

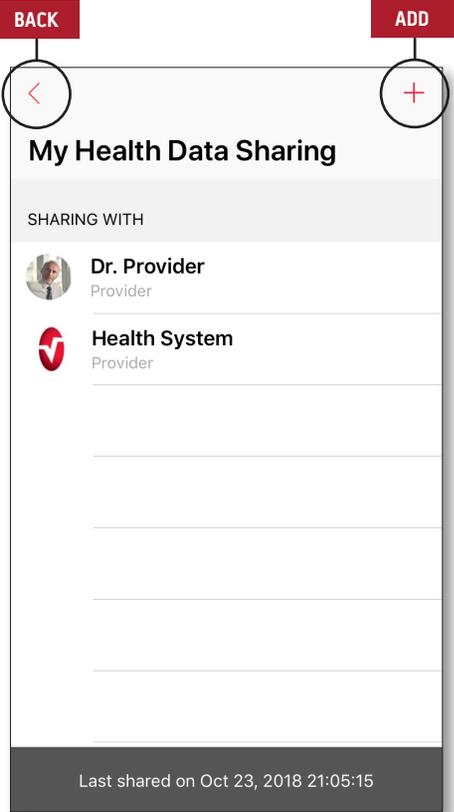

- > **Displays the list of shared healthcare providers, caregivers, and family members.**
- > **Add**
 - Share with additional healthcare providers.
 - Share with additional caregivers or family members.
- > **Swipe to edit privileges.**

ADD

