

Masimo SafetyNet®

NIBP Setup Instructions



Get to Know Your System

Masimo SafetyNet App

Masimo SafetyNet displays your NIBP* measurements. It also allows data updates to your care provider.

*NIBP is short for **non-invasive blood pressure**

WARNING: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.




Omron Blood Pressure Monitor

(refer to HEM-9200T Quick Start Guide)



Getting Started

- 1 Open invite from your care provider.**
In the email click on the link to download the Masimo SafetyNet App. Then open the app and sign up to create your account.
- 2** Once you have created and logged in to your account go to the menu and click on "My Devices". Then select your NIBP device to connect. Follow the on-screen instructions in the app to pair to the Omron NIBP device.

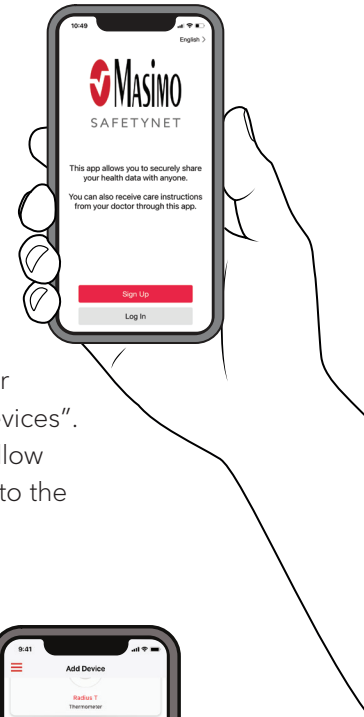


- 3 Take your first blood pressure measurement**
Follow the instructions in the app showing you how to take a measurement.
(See back for details on taking a measurement.)

For more info go to: [mymasimo.com](https://www.mymasimo.com)




How to Apply Your Blood Pressure Cuff and Take a Measurement


1 Connect cuff plug to the unit.

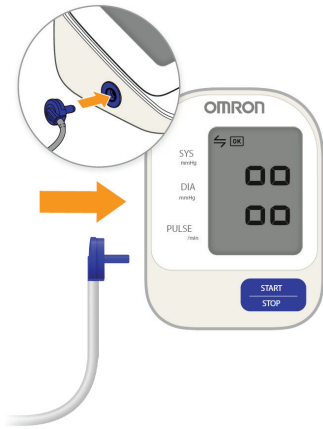

2 Wrap the arm cuff firmly in place around your left upper arm. Secure closed with the fabric fastener.

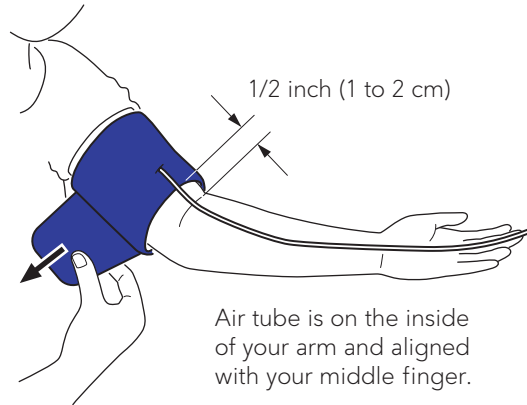


3 Press the START/STOP button. The arm cuff will start to inflate automatically.

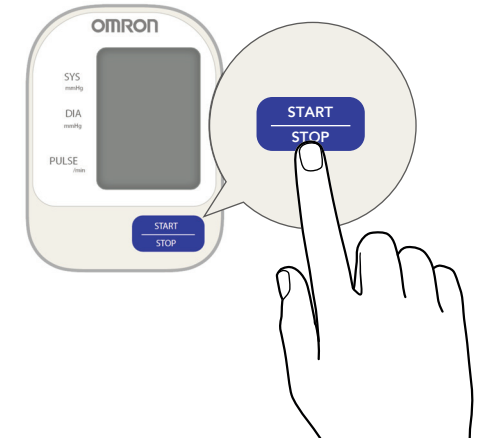

Tips for Taking a Measurement

- Refer to Omron HEM-9200T instruction manual
- Seat relaxed and comfortably under room temperature.
- The arm cuff should be placed on your arm at the same level as your heart.
- Remove tight-fitting clothing.
- Be careful not to rest your arm on the air tube.
- Always use the same arm for measurement.

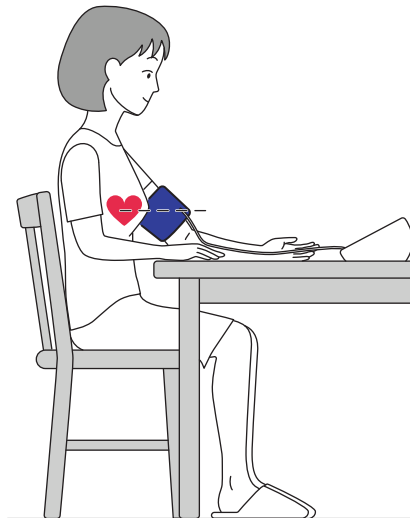

How to Find Your Measurement History

- Tap on "Dashboard" on home screen and then tap on your blood pressure chart on the dashboard.

or

- Tap on Notification.
* make sure **Notifications** are enabled.

