

Masimo SafetyNet®

Scale Setup Instructions



Get to Know Your System

Masimo SafetyNet App

Masimo SafetyNet displays your weight scale measurements. It also allows data updates to your care provider.

WARNING: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.





Omron Digital Weight Scale

(refer to HN-290T Quick Start Guide)



Getting Started

- 1 Open invite from your care provider.**
In the email click on the link to download the Masimo SafetyNet App. Then open the app and sign up to create your account.
- 2** Once you have created and logged in to your account go to the menu and click on "My Devices". Then select your weight scale to connect. Follow the on-screen instructions in the app to pair to the Omron weight scale.

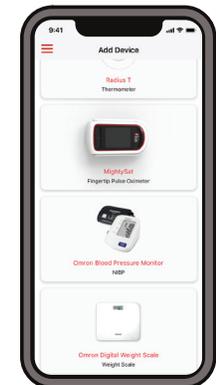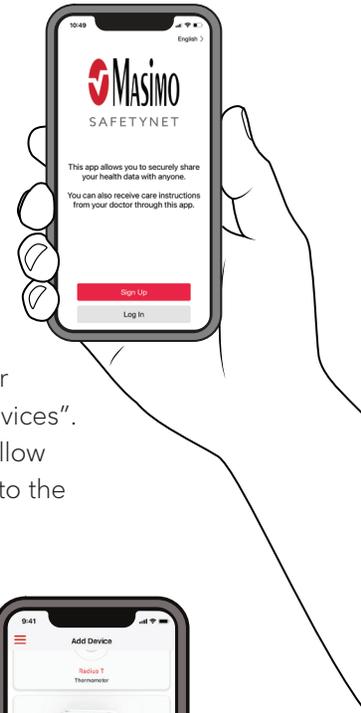

- 3 Take your first weight measurement**
Follow the instruction in the app showing you how to take a measurement.
(See back for details on taking a measurement.)


For more info go to: mymasimo.com

see back for scale set up ►




How to Set Up Your Weight Scale and Take a Measurement

- 1** Select the weight metric "kg" or "lb" when the power is off.

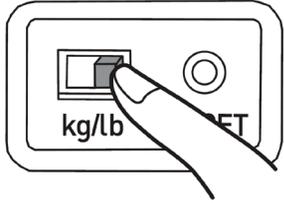

- 2** Remove the battery cover on the back of the unit.
Insert 4 "AA" batteries as indicated in the battery compartment.

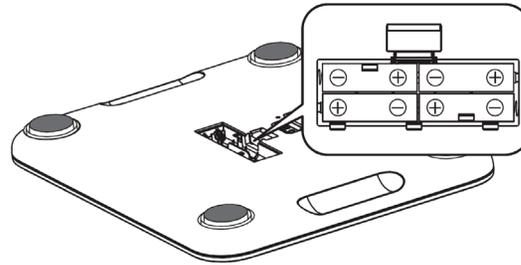


- 3** Step onto the unit.
When you step onto the unit, the power automatically turns on.
When the unit beeps twice, the measurement is completed.



Tips for Taking a Measurement

- Refer to Omron HN-290T instruction manual.
- Do not place the unit on a cushioned surface such as a carpet or a mat.
- Try to measure your weight at the same time each day for consistency.

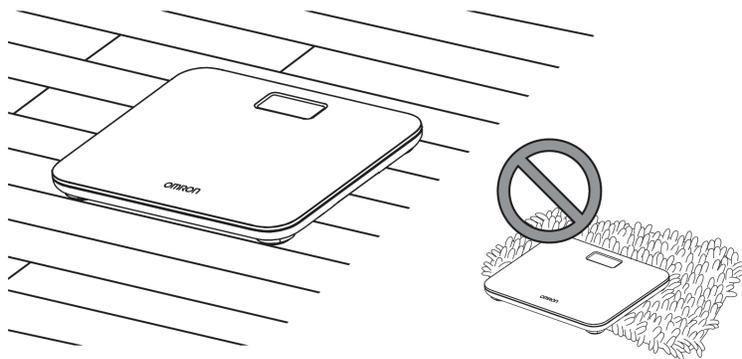

How to Find Your Measurement History

- Tap on "Dashboard" on home screen and then tap on your weight chart on the dashboard.

or

- Tap on Notification.
* make sure **Notifications** are enabled.

