

Masimo SafetyNet®

TIR-1 Setup Instructions



Get to Know Your System

Masimo SafetyNet App

Masimo SafetyNet displays your temperature measurements. It also allows data updates to your care provider.

WARNING: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.




TIR-1® Non-Contact Thermometer



Getting Started

- 1 Open invite from your care provider.**
In the email click on the link to download the Masimo SafetyNet App. Then open the app and sign up to create your account.
- 2** Once you have created and logged in to your account go to the menu and click on “My Devices”. Then select your TIR-1 device to connect. Follow the on-screen instructions in the app to pair to the TIR-1 device.

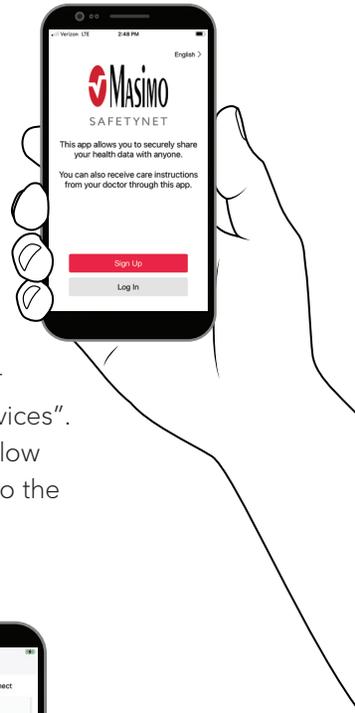


- 3 Take your first temperature measurement**
Follow the instructions in the app showing you how to take a measurement.
(See back for details on taking a measurement.)

For more info go to: [mymasimo.com](https://www.mymasimo.com)




How to Take a Measurement


1 Remove the protective cap.

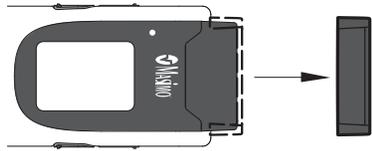

2 Clean the central area of forehead.

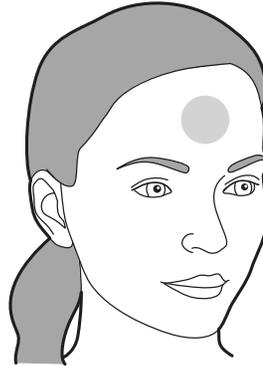


3 Aim at middle of forehead, 1/2 – 2 inches away from skin. Press and release the POWER button. Hold thermometer in place until 2 beeps are heard and temperature is displayed on LCD.

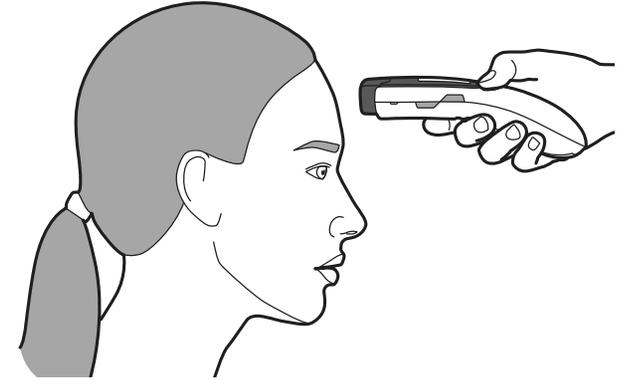


Replacing Batteries

1 Press down on top section of battery cover and slide in the direction of the arrow on the cover.

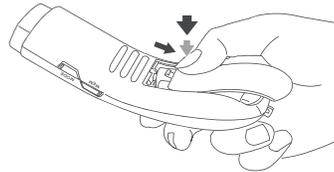

2 Insert 2 AA Alkaline batteries in the compartment and wait for the beep and the LCD to light up.

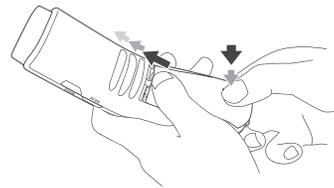

3 Slide battery cover back over the compartment with pressure on both top and bottom of the battery cover.



How to Find Your Measurement History

• Tap on "Dashboard" on home screen and then tap on your temperature chart on the dashboard.

or

• Tap on Notification.

* make sure **Notifications** are enabled.

